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ENG USER INSTRUCTION

Construction

The protective face shield OTA-1 consists of the impact shield and bracing band with regulation that you can adjust to fit different head sizes.

The shield is made of polycarbonate:

- ✓ one of the most transparent materials among plastics,
- ✓ resistant to disinfecting substances (including those containing alcohol).

Requirements

The protective face shield meet the requirements of: EN166:2001 "Personal eye-protection — Specifications" and Regulation (eu) 2016/425 of the European Parliament and of the Council of 9 March 2016.

The marking and meaning of used symbols

The protective face shield are marked as: AQUAEL OTA-1 EN166:2001 3 CE

The panes as a replaceable element are marked: AQUAEL 1 CE

The meaning of used symbols:

- ✓ AQUAEL - manufacturer's identification mark,
- ✓ OTA-1 - protective face shield type,
- ✓ EN166: 2001 - number of product standard, EN166:2001 - "Personal eye-protection — Specifications",
- ✓ 3 - protection against liquid splashes,
- ✓ 1 - optical class
- ✓ CE - certification mark confirming compliance with the requirements specified in the Regulation of the European Parliament and of the Council (EU) 2016/425 of March 9, 2016.

Usage

It is used to:

- ✓ protect against splashing liquids,
- ✓ protect the mucous membranes of mouth, eyes and nose from contact with contaminated hands, objects and from minor mechanical injuries.

Assembly

Method of assembly has been presented in the pictures 1-5.

It is possible to adjust the protective face shield to fit different head sizes thanks to a bracing band with regulation (picture 5). Remove the protective foil (applied on both sides).

Protective and functional properties

Protective face shield meet the requirements of EN166:2001 (PN-EN166:2005) in the scope of:

- ✓ resistance to inflammation and elevated temperature,
- ✓ optical parameters.

The optical quality of the lens in protective face shield is in accordance with the requirements for the first optical class. The cover is suitable for continuous work.

The materials of which the protective face shield are made, do not cause allergic reactions in contact with the user's skin.

Each item is packed in a plastic bag, and during storage or transport they are packed in cartons.

Usage, storage and maintenance

Protective face shield must be protected against scratches and kept clean. There should be no dirt, dust, streaks or fingerprints on the surface of the lens. The clarity should be checked in the light against a dark background. In order to clean the protective face shield, wash it in water with commonly used dishwashing liquids, then rinse and leave to dry. A damaged or excessively scratched visor should be replaced. The protective face shield can be used indefinitely. The product should be stored in such a position that the lenses are with a convex surface upwards - without making contact with surrounding objects.

NOTE: A cracked, scratched or damaged product must be replaced.

The EU declaration of conformity is available at: www.aquael.suwalki.pl

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